

Protocol for single layer centrifugation of Porcine sperm


Materials required

- **PorciPure™** BottomLayer
- **PorciExtend™**
- **PorciWash™**
- Conical centrifuge tubes
- Dispensing pipette and disposable tips
- Pasteur pipettes
- Centrifuge with swing-out rotor

Protocol

Depending on the volume ejaculate to process, different sizes of tubes can be used. See table below.

Ejaculate size	Tube size (mL)	Bottom layer Volume (mL)	Ejaculate volume (mL) on the gradient
Small	10-15	4	1,5
Medium	50	10	3-4
Larger	50	20	7-8

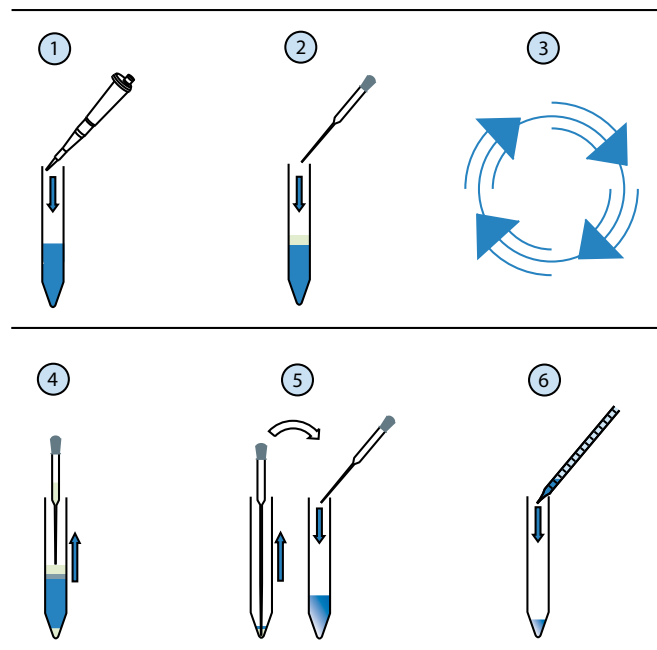

- Bring all materials to room temperature.
 - Extend the ejaculate with **PorciExtend™** (max conc 250 million/mL).
1. Using a sterile pipette, transfer **PorciPure™** Bottom Layer to a conical centrifuge tube (see volumes above).
 2. Using a new sterile pipette, layer extended semen on top of the **PorciPure™** Bottom layer.
 3. Centrifuge at 300 g for 30 minutes at room temperature in a centrifuge with a swing-out rotor. Do not use the brake.
 4. Carefully remove ejaculate and most of the **PorciPure™** Bottom Layer.
 5. Using a **new** sterile pipette, transfer and resuspend sperm pellet in 1 mL **PorciWash™** in a new sterile tube (10-15 mL).
 6. Dilute to desired concentration with **PorciWash™**.

Note: we recommend preparing two **PorciPure™** single layer gradients for each sample, to reduce the risk of overloading a single gradient and to provide two tubes to balance the centrifuge rotor.